

# USERS      MANUAL



**HP7S**




**HEARTWAY MEDICAL PRODUCTS CO., LTD.**

Part no:70030063

## COMPONENTS

Your power wheelchair is shipped partially disassembled to protect it during shipping. After unpacking, please check whether you have received the following main components as our standard specification (See Fig.1).

- |                                 |                                |
|---------------------------------|--------------------------------|
| <b>1. Main frame with motor</b> | <b>7.Armrest (left)</b>        |
| <b>2. Reclining back set</b>    | <b>8.Footrest frame(right)</b> |
| <b>3. Seat</b>                  | <b>9.Footrest Frame(left)</b>  |
| <b>4. Armrest frame (right)</b> | <b>10.Footrest (right)</b>     |
| <b>5. Armrest frame (left)</b>  | <b>11.Footrest(left)</b>       |
| <b>6. Armrest (right)</b>       | <b>12.Controller</b>           |

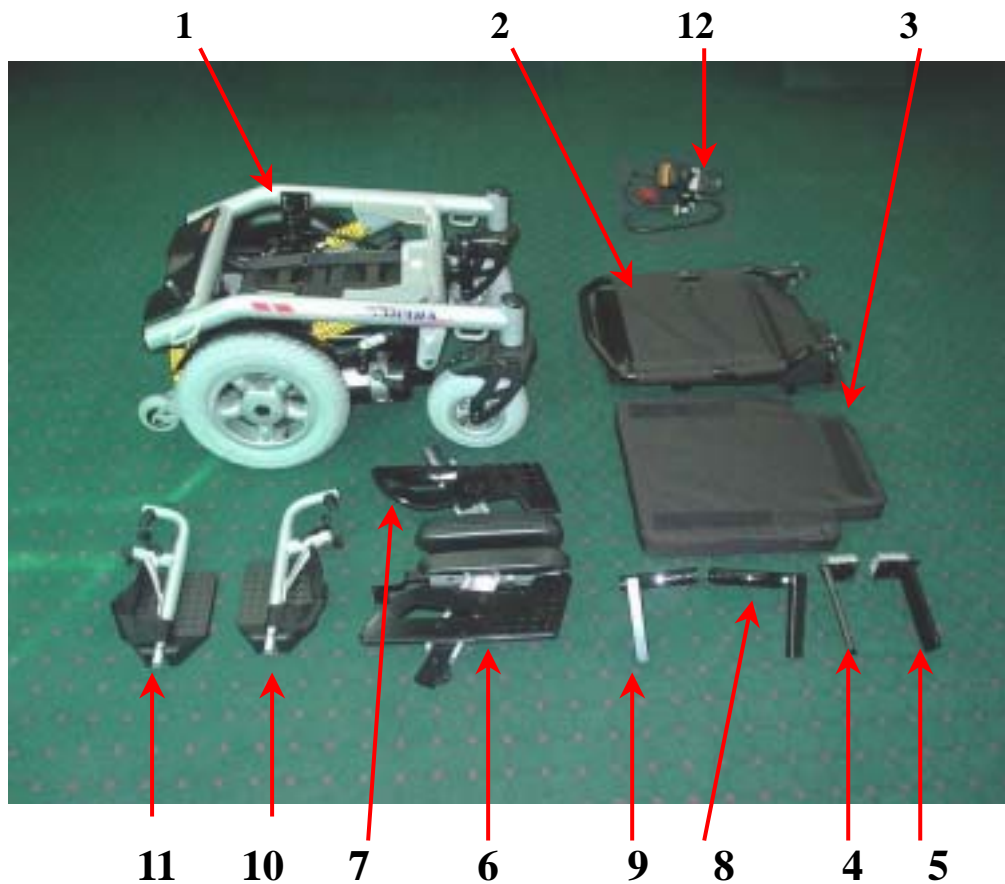

(Fig 1)

## **SAFETY INSTRUCTION**

### **Operation of Chair**

1. Always ensure that the power is switched off when getting in or out of the wheelchair. This will eliminate the possibility of accidentally activating the joystick and causing injury to yourself or others.
2. Always check that the drive wheels are engaged (drive mode) before driving.
3. Set the speed control knob according to your driving ability and the environment in which you are going to operate. We recommend that you keep your speed at the slowest position (fully press the deceleration button) until you are familiar with the driving characteristics of the vehicle. We also recommend that you use the slowest speed when using your power wheelchair indoors.
4. Always reduce your speed when making sharp turns.
5. Do not switch off the power when the wheelchair is still moving forward. This will bring the chair to an extremely abrupt stop.
6. Avoid jerky stop/start motions as they will result in excessive current draw from the batteries, increased tire wear and the rapid wearing of the gear boxes and motors.
7. To brake in an emergency, simply release the joystick.

### **Ramps and Curbs**

8. When driving up or down ramps, be sure to check that the angle of the slope is less than 10 degrees (slopes about 1/6). Also check that ramp surface is roughened to prevent slipping. Never drive across the slope or turn sharply on a slope.
9. When driving up curbs, always check the height of the curb to ensure that it does not exceed 1-1/2" height.

### **Transfers, Reaching and Bending**

10. Transferring on and off the HP7S requires a good sense of balance. To eliminate the possibility of injury, we recommend performing the following tasks before attempting a transfer:

Position chair so that the distance between your power chair and the object to which you are transferring is close enough for a safe transfer.

Turn the power off

Ensure that your power chair is not in freewheel mode.

Flip up or remove armrests

Flip up footplate or remove footrests

Turn both caster wheels towards the transfer direction to improve power chair stability during transfer.

11. When reaching, bending or leaning while seated on your power chair, make sure that you maintain a stable center of gravity to keep the power chair from tipping.

### **General**

12. Always use a seat belt, and keep feet on the footplate at all the time.
13. For safety reasons, make sure that your weight does not exceed the recommended weight limit of the wheelchair. Consult your dealer for the specified weight limits for your particular model.
14. Do not attempt to lift or move a power chair by any of its removable parts. Personal injury and damage to the power chair may result.
15. Do not stand on the footplate directly.
16. Never try to use your wheelchair beyond its limitations as described in this manual.
17. Do not operate your vehicle if it is not functioning properly.
18. Do not connect any electrical or mechanical device to the vehicle. Failure to obey this instruction may result in injury and will void the warranty.
19. Never use electronic radio transmitters such as CB, walkie-talkies, portable computers or cellular phones while using the vehicle without first turning the vehicle off.

### **Use While Under The Influence Of Medication Or Alcohol**

20. Check with your physician if you are taking any medication that may affect your ability to operate your power wheelchair safely.
21. Do not operate your vehicle while you are under the influence of alcohol, as this may impair your ability to operate your power chair in a safe manner.

### ***Electromagnetic interference (EMI) from Radio Wave Sources***

The rapid development of electronics, especially in the area of communications, has saturated our environment with electromagnetic(EM) radio waves that are emitted by television, radio and communication signals. These EM wave are invisible and their strength increases as one approaches the source. All electrical conductors act as antennas to the EM signals and, to varying degrees, all power wheelchairs and scooters are susceptible to electromagnetic interference(EMI). The interference could result in abnormal, unintentional movement and/or erratic control of the vehicle. The United States Food and drug Administration (FDA) suggests that the following statement be incorporated to the user's manual for all power wheelchairs like the HP-7S. Power wheelchairs and motorized scooters (in this section, both will be referred to as powered wheelchairs) may as susceptible to electromagnetic interference (EMI),

which is interfering electromagnetic energy emitted from sources such as radio stations, TV stations, amateur radio (HAM) transmitter, two-way radios and cellular phones. The interference (from radio wave sources) can cause the powered wheelchair to release its brakes, move by itself or move in unintended directions. It can also permanently damage the powered wheelchair's control system. The intensity of the EM energy can be measured in volts per meter(V/m).Each powered wheelchair can resist EMI up to a certain intensity. This is called "immunity level". The higher the immunity level, the greater the protection. At this time, current technology is capable of providing at least 20 V/m of immunity level, which would provide useful protection against common sources of radiated EMI.

Following the warnings listed below should reduce the chance of unintended brake release or powered wheelchair movement that could result in serious injury:

- 1,Do not turn on hand-held personal communication devices such as citizens band(CB) radios and cellular phones while the powered wheelchair is turned on.
- 2.Be aware of nearby transmitters such as radio or TV stations and try to avoid coming close to them.
- 3.If unintended movement or brake release occurs, turn the powered wheelchair off as soon as it is safe.
- 4.Be aware that adding accessories or components, or modifying the powered wheelchair, may make it more susceptible to interference from radio wave sources (Note: It is difficult to evaluate the effect on the overall immunity of the powered wheelchair).
- 5.Report all incidents of unintended movement or brake release to the powered wheelchair manufacturer, and note whether there is a radio wave source nearby.

#### TURN OFF YOUR POWERED WHEELCHAIR AS SOON AS POSSIBLE WHEN EXPERIENCING THE FOLLOWING:

- Unintentional motions
- Unintended or uncontrollable direction.
- Unexpected brake release

The FDA has written to the manufacturers of power wheelchairs asking them to test new products to be sure they provide a reasonable degree of immunity against EMI. The FDA requires that a powered wheelchair should have an immunity level at least 20 V/m, which provides a reasonable degree of protection against more common sources of EMI. The higher the immunity level, the greater the protection. Your powered wheelchair has an immunity level of 20 V/m which should protect against common sources of EMI.

## ENVIRONMENTAL CONDITIONS

Environmental conditions may affect the safety and performance of your power wheelchair. Water and extreme temperatures are the main elements that can cause damage and affect performance.

### A) Rain, Sleet and Snow

If exposed to water, your power wheelchair is susceptible to damage to electronic or mechanical components. Water can cause electronic malfunction or promote premature corrosion of electrical components and frame.


### B) Temperature

Some of the parts of the power wheelchair are susceptible to change in temperature. The controller can only operate in temperature that range between 18 (-8 ) and 122 (50 ).

At extreme low temperatures, the batteries may freeze, and your power wheelchair may not be able to operate. In extreme high temperatures, it may operate at slower speeds due to a safety feature of the controller that prevents damage to the motors and other electrical components.

## ASSEMBLY INSTRUCTION


### 1.Installing the Seat (#3) and Reclining Back(#2)

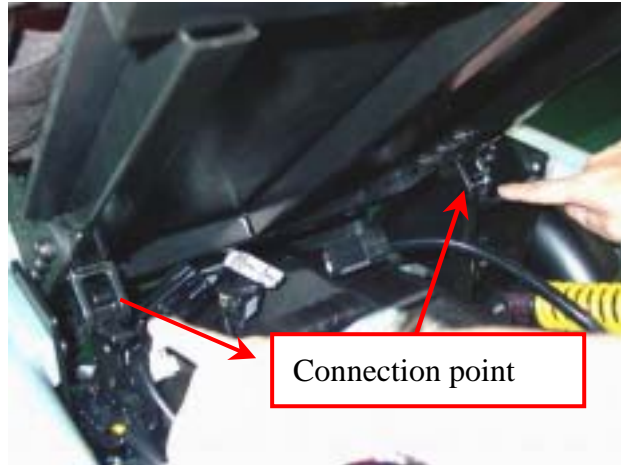

(Fig 2)

Slide the reclining back set (#2) into the Connection points (both sides) in the Main frame (#1)(See Fig 2), put down the reclining back set and connect the front connection point and make sure that tighten the tension knob in the front connection point. (See Fig3)

Put the seat on the reclining back set in the right direction. (See Fig 4)

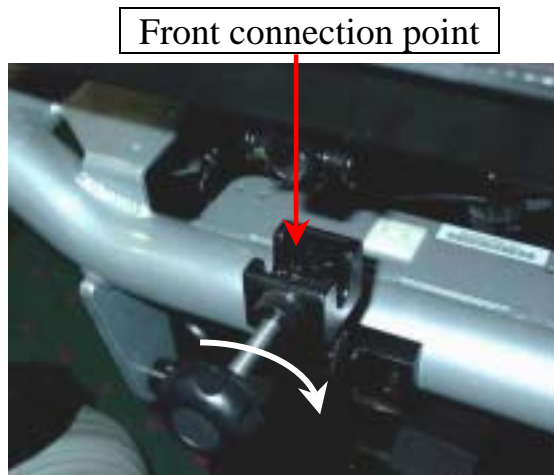

(Fig 3)




(Fig 4)

Adjust the comfortable back seat angle and insert the spring pin into the desired location and let the spring to attach the other end. (See Fig5)

## 2.Installing the Armrest Frame

Put the armrest frame ( with left-#5 and Right-#4 each) into the main frame and make sure tighten the bolts with Allen key(No#4). (See Fig 6)

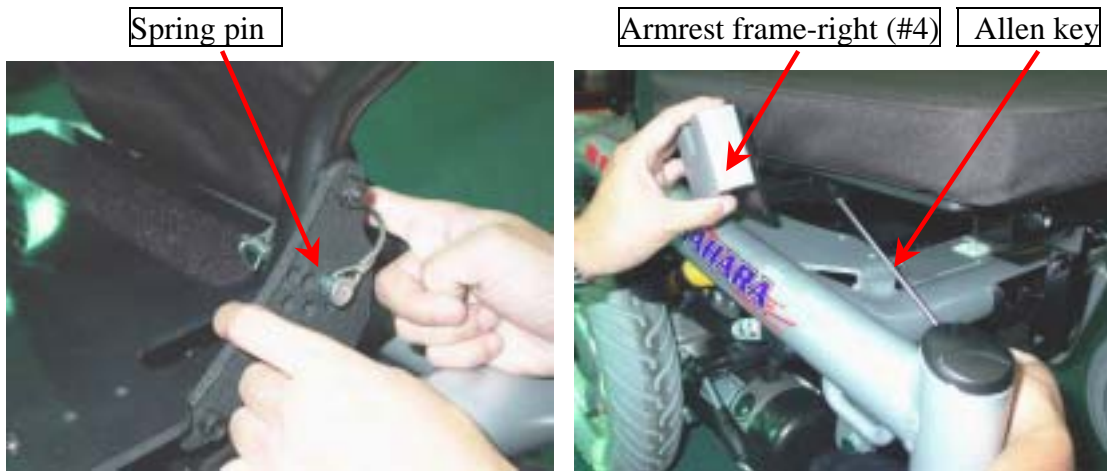

(Fig 5)

(Fig 6)

## 3.Installing the Armrest

Insert the armrest (with left-#7 and Right-#6 each) into armrest frame then use the tension knob to lock the armrest. (See Fig 7)

## 4. Installing the footrest frame

Put the footrest frame (with left-#9 and Right-#8 each) into the main frame and make sure tighten the bolts with No #4Allen key.(See Fig 8)



(Fig 7)

(Fig 8)

## 5.Installing the footrest

Insert the footrest( with left-#11 and Right-#10 each) into footrest frame with side direction(See Fig 9) then turn around 90 degree to correct direction (See Fig 10) and make sure the pin jumping into the correct position. (See Fig 11)



(Fig 9)



(Fig 10)




(Fig 11)

### 6.Installing controller and connect cables

Assemble the controller(#12) on right armrest (#6) bracket with 2 screws.(See Fig 12)  
There is 1 plug (9 pins) in the other end on the controller. Please plug in the main frame (#1) plugs and make sure tightly connection in each plugs.(See Fig 13)




head screw driver (Fig 12)

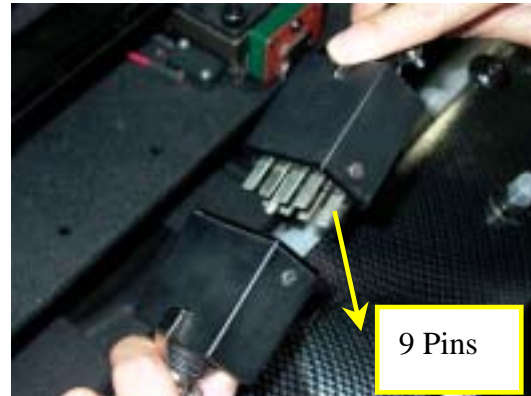

(Fig 13)

Note: Besides you have to plug in the main frame for batteries cables (connect for color by color) if you will reconnect your batteries cable. (See Fig 14)

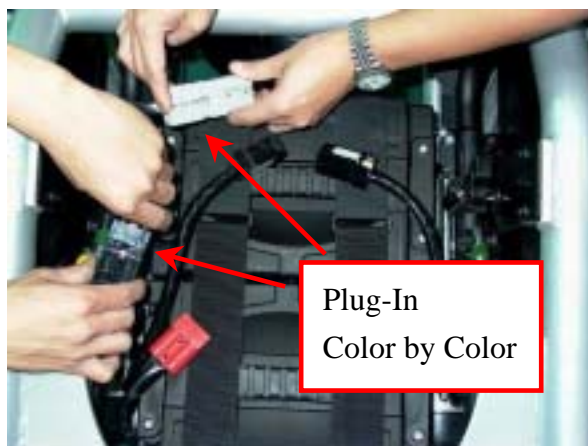

(Fig 14)

## ADJUSTMENTS FOR SEATING CONFORT

To maximize seating comfort, your power wheelchair lets you adjust:

Armrest height and width

Footplate width

Controller position

Backrest angle

### A. Armrest Height and Width Adjustment

A- 1: To adjust the Height:

loosen the T-Type knob

slide the armrest in or out for desired height

tighten the T-Type knob

A- 2: To adjust the Width:

use Allen key to loosen armrest frame

slide the armrest frame in or out for desired width

tighten bolt with Allen key (No #4)



(Fig 15)



(Fig 16)

### B. Footplate Width Adjustment

use Allen key to loosen footrest frame

slide the footrest frame in or out for desired width

tighten bolt with Allen key

### C. Controller Position Adjustment

Loosen the T-knob (under the armrest)

pull out the whole set controller to your desired length

tighten the T-knob



(Fig 17)



(Fig 18)

#### D. Backrest Angle Adjustment


- move the backrest to your comfortable angle
- insert the spring pin into the desired hole location
- let the spring attach the other end of the pin.

#### Note:

There are 7 different angles when you adjust the backrest to fit yours comfortable seating from the 90° to 135°. (See Fig 20)



(Fig 19)

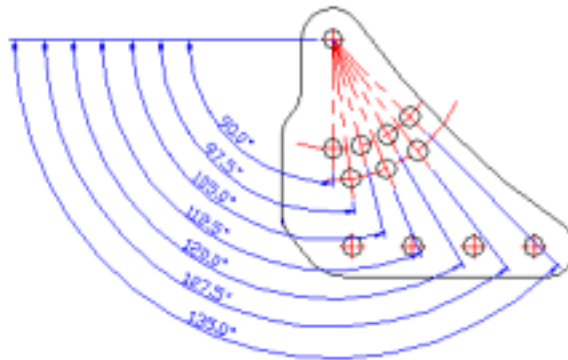

(Fig 20)


## OPERATION

The power wheelchair is simple to operate. However, we recommend that you read carefully the following instructions to become familiarized with your new vehicle.

### A Word of Caution:

Before you turn the power on, always be aware of the environment that surrounds you to select your desired speed. For indoor environments we recommend that you select the slowest speed setting. For outdoor operation of this vehicle we recommend that you select a speed that is comfortable for you to control it safely.

The following steps are required to operate your vehicle safely with the controller (See Fig 21)

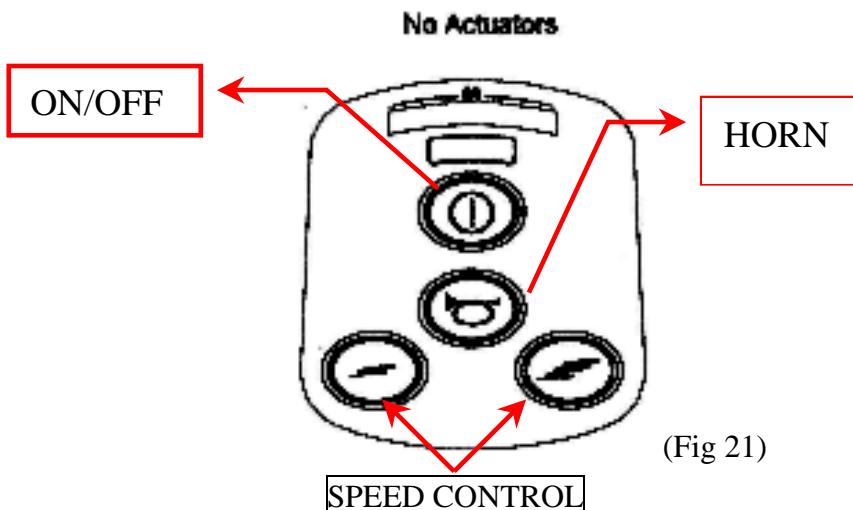

(Fig 21)

### A. Driving:

#### 1. Controller ON/OFF Switch

Press the ON/OFF button (I/O) switch located in front of the joystick to activate your power wheelchair. The battery condition meter will light up to indicate the current charge of your battery. Pressing the ON/OFF button again will deactivate the controller.

#### 2. Speed Control

The speed control buttons allow you to set the forward speed to your desired setting. Pressing the speed increasing button will set the speed at the faster settings while the speed will slow down if you press the speed decreasing button. The controller sets the reverse speed, acceleration and deceleration proportionally and automatically for your safety.

### 3. Joystick

The joystick controls the direction and speed of your vehicle, pointing the joystick away from the neutral position, (center), will move the vehicle in the direction where the joystick is pointing. The farther away (forward/backward) the joystick is from the neutral position, the faster the vehicle will go. The farther away to the right/ left the joystick is pointing, the sharper the turn of the vehicle will be. To operate the vehicle gently push the joystick in the direction you want to go. Returning the joystick to its neutral position, (center), will reduce the speed and stop the vehicle by automatically applying the electromagnetic brakes.

#### Notes:

After pressing the controller ON/OFF switch, allow two seconds to elapse before engaging the joystick. This is a safety feature to prevent sudden starts. Gentle operation of the joystick will result in smoother transitions in speed and direction, while sharp operation of the joystick will result in drastic transitions in direction and velocity.

When the wheelchair is in operation, the surface of the charger will become slightly hot.

In case of emergency, let go of the joystick and the chair will come to a stop.

### B. Controller Display

The controller display is a multifunction visual display. It can provide four types of information: 1) ON/OFF status, 2) battery condition meter, 3) Horn, 4) acceleration and deceleration buttons.

#### 1. N/OFF Status

When the power is on, the controller's LED will be lit up. If the LED is not lit, the controller is OFF.

#### 2. Battery Condition Meter

The battery condition meter is composed of 10 segments (three of red, four of yellow, and three of green). It enables you to monitor battery charge. The battery condition meter indicates the approximate amount of battery charge left.

Red, yellow and green LEDs indicate that the batteries are fully charge

Red and yellow LEDs indicate that you should charge the batteries if possible.

Red LEDs indicate that you should charge the batteries as soon as possible, because low battery voltage may cause your power wheelchair to become inoperative.

**Note:** When the batteries begin to approach a discharged state, the first red LED will begin to flash slowly, reminding you that the batteries need to

be charged immediately.

### C. Free-Wheeling:

Because the motors are designed to engage the electromagnetic brakes when the vehicle is not in use or when the power is OFF, they also have a manual feature that allows them to “free-wheel”. Free-wheeling is accomplished by turning the free-wheeling levers to the free-wheeling position.(See Fig 22)



(Fig 22)

#### **Warning !**

Never free-wheel your power wheelchair on a slope.

Never free-wheel the motors while operating your vehicle.

Always remember to engage the motors before turning the power back ON.

### D. Electromagnetic Brakes:

Your power wheelchair comes with an Electromagnetic Brakes., i.e. an automatic magnetic disc safety brake which is also known as Fail-Safe brake.

The Electromagnetic Brakes are automatic and work when the power wheelchairs is ON but in a steady state (i.e. Joystick is released to the neutral position), even when the chair is on a slope. The Electromagnetic Brakes will also be set whenever the power wheelchair is OFF, but the motor levers are in the engaged (vertical) position.


Note: Please refer to the section titled to check brake in the Maintenance & Repair section in page 18 to make sure brakes are in good condition.

### E. Thermal Protection:

Your power wheelchair controller is equipped with a safety system called thermal rollback. A built-in circuit monitors the temperature of the controller and motors, the controller reduces the motor voltage and speed of the power wheelchair. The reduction of the speed allows the electrical components to cool down. Although your power wheelchair will resume its normal speed when the temperature returns to a safe level, we recommend that you turn the power off and wait for 5 minutes before restarting to allow the components to cool down if you find that you have lost speed suddenly.

F. Main Circuit Breaker:

The main circuit breakers reset button located near the seat.(Fig 23 )

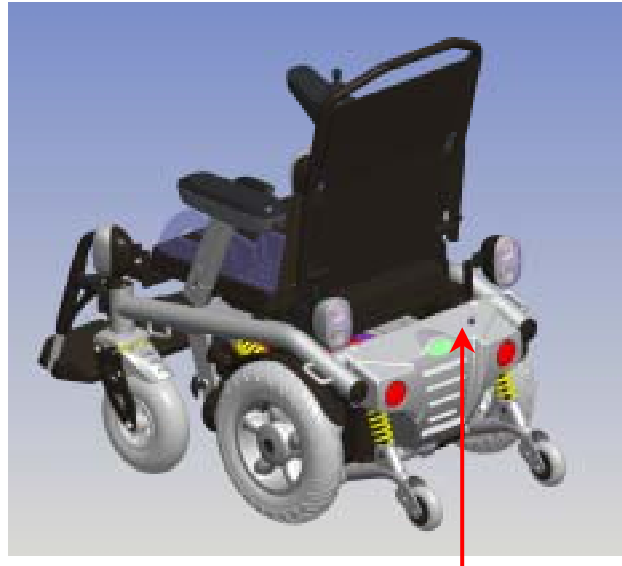

(Fig 23)

Breaker

The main circuit breaker monitors the electric current drawn from the battery. It is a safety feature built in your power wheelchair for your extra safety. When the batteries and motors are heavily strained (e.g., from excessive loads), the main circuit breaker will trip to prevent damage to the motor and the electronics. If the circuit breaker trips, wait for approximately one minute and then depress the button to reset it. Then turn on the controller power, and continue normal operation. If the main circuit breaker continues to trip repeatedly, contact your authorized dealer.

## BATTERIES & CHARGER

### **BATTERY**

We recommend that you use deep-cycle batteries that are sealed and maintenance free for your power wheelchair. Both sealed lead-acid (SLA) and gel cell are deep-cycle batteries and are similar in performance. Deep-cycle batteries are specifically designed to provide power, drain down, and then accept a relatively quick recharge. Lead-acid batteries should be charge as often as possible.

Specification of the battery that we recommend:

Type:	Deep –cycle sealed lead-acid or gel cell
Size:	22NF
Voltage:	12V each
Amp Hours:	50-62 amp hours

Depending on the use, terrain and driving conditions, the batteries will provide a range of 12-18 miles of travel. However, even if the power chair is not in use, we recommend that the batteries be charged periodically.

Note:

Do not use any automotive batteries. They are not designed to handle a long, deep discharge and also are unsafe for use in power chairs.

The useful life of a battery is quite often a reflection of the care it receives.

### **CHARGER**

The battery charger takes the standard wall outlet voltage (alternating current )and converts it into VDC (direct current).The batteries use direct current to run your power chair. When the batteries are fully charged, the amperage from the charger is almost at zero. This is how the charger maintains a charge but does not over-charge the battery.

Note: The batteries will not be able to be charged after they are discharged to nearly zero voltage.

### **CHARGING INSTRUCTIONS**

To recharge the batteries, follow the steps below:

Place your power wheelchair close to a standard electrical wall outlet.

Turn the controller power OFF

Plug the charger power cord (Round head) into the controller

Plug the charger power cord into a standard wall outlet. A red and a yellow (if battery charge is low) LED will illuminate.

When charging is completed, the yellow LED will turn green.

Disconnect the charger power cord from the wall outlet when the batteries are fully charged. (See Fig 24)

**Important!**

Do not use for voltage input except the specified. Make sure your present voltage input (110V or 220V) and adjust manually. (See Fig 25 )



plug-in controller    plug-in wall outlet  
(Fig 24)



(Fig 25)

**Note:**

Always charge your batteries in well ventilated areas.

The charger is intended for indoor use only. Protect from moisture.

For maximum performance , it is recommended that you replace both batteries at the same time if the batteries are weak.

If the chair will not be used for a long period of time, arrange to have the batteries fully charge for at least once every month.

According to the battery type and condition of the batteries, batteries usually can be fully charged in 4-10 hours. This will be indicated when the status light in the battery charger side panel turns green. Charging the battery longer than necessary will not harm the battery. We recommended that you charge the batteries for 8 to 10 hours after daily use. Do not charge the batteries for more than 24 hours.

**Note:** There is a battery circuit diagram labeled on the frame. Please refer this diagram before you assemble the battery.

## MAINTENANCE & REPAIR

Your power wheelchair is designed for minimal maintenance. However, like any motorized vehicle it requires routine maintenance.

To keep your HP7S for years of trouble-free operation, we recommend you follow the following maintenance checks as scheduled.

### DAILY CHECKS

1. With the controller turned OFF, perform a joystick check. Make sure it is not bent or damaged and that it returns to center when you release it. Visually inspect the rubber boot around the base of the joystick for damage. Do not handle or try to repair it.
2. Visually inspect the controller harnesses. Make sure that they are not frayed, cut or have any exposed wires.
3. Inspect the battery condition meter on the controller to determine if batteries need to be charged.

### WEEKLY CHECKS

1. Inspect the connections by disconnecting the controller and charger harnesses from the electronics connector housing. Look for signs of corrosion.
2. Ensure that all parts of the controller system are securely fastened to your power wheelchair. Do not over tighten any screws.
3. Check for proper tire inflation. Your power wheelchair comes with standard flat-free solid tires. If your power wheelchair come with optional air tires, make sure to maintain the pressure of the tires between 30-35 psi.
4. Check the brakes. This test should be carried out on a level surface with at least three feet of clearance around your power wheelchair.

To check the brakes ( Your power wheelchair may move slightly when performing this test):

Turn on the controller and turn down the speed and response adjustment knob. After one second, check that the battery condition meter remains on condition. Slowly push the joystick forward until you hear the parking brakes click. Immediately release the joystick. You must be able to hear each parking brake operate within a few seconds of joystick release.

Repeat this test of the brake for the back, left and right joystick positions

### SEMI-ANNUAL CHECKS

1. Check the motor brushes. We recommended that your authorized dealer inspect the bushes every six months, or sooner if your power wheelchair is not operating

smoothly. If inspection determines excessive wear on the brushes, they must be

replaced or motor damage will result.

**Warning! Failure to maintain the brushes could void the power wheelchair warranty.**

To inspect or replace the motor brushes:

1. Unscrew the motor brush caps. (See Fig 26)
2. Remove the brushes.
3. Inspect the brushes for wear. (See Fig 27)
4. Replace the brushes if necessary.



New Motor Brush      Worn Motor Brush

(Fig 26)



Motor Brush caps

(Fig 27)

2. Inspect the state of the battery terminals every six months. Make sure that they are not corroded and the connections are tight. Periodically apply a thin film of petroleum jelly on the surface of terminals to guard against corrosion.

#### PERIODICAL CHECKS

1. Make sure to keep the controller clean while protecting it from rain or water. Never hose off your power wheelchair or place it in direct contact with water.
2. Keep wheels free from lint, hair, sand and carpet fibers.
3. Visually inspect the tire tread. If less than 1/32", please have your tires replaced by your local dealer.
4. All upholstery can be washed with warm water and mild soap. Occasionally check the seat and back for sagging, cuts and tears. replace if necessary. Do not store your chair in damp conditions as this will lead to mildew and rapid deterioration of the upholstery parts.
5. All moving mechanism will benefit from simple lubrication and inspection. Lubricate using petroleum jelly or light oil. Do not use too much oil, otherwise small drips could stain and damage carpets and furnishings etc. Always perform a general inspection of the tightness of all nuts and bolts.

Note:

If you experience any technical problems, it is recommended that you check with your local dealer before attempting to troubleshoot on your own.

The following symptoms could indicate a serious problem with your power wheelchair. Contact your local dealer if any of the following arises:

1. Motor noise
2. Frayed harnesses
3. Cracked or broken connectors
4. Uneven wear on any of tires
5. Jerky motion
6. Pulling to one side
7. Bent or broken wheel assemblies
8. Does not power up
9. Powers up, but dose not move